

ETV6/NTRK3 t(12;15) Gene fusion Probe Detection Kit (CW-083)

Intended use

This kit performs in situ hybridization staining on the basis of conventional staining to provide physicians with auxiliary information for diagnosis. The test results are for clinical reference only and should not be used as the only basis for clinical diagnosis. Clinicians should make comprehensive judgment on the test results based on factors such as the patient's condition, drug indications, treatment response and other laboratory test indicators.

Product composition

The kit consists of ETV6/NTRK3 dual color probe (100µL/Tube).

Storage condition

Keep sealed away from light at $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$. The product is valid for 12 months. Avoid unnecessary repeated freezing and thawing that should not exceed 10 times. After opening, within 24 hours for short-term preservation, keep sealed at $2-8^{\circ}\text{C}$ in dark. For long-term preservation after opening, keep the lid sealed at $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$ away from light.

Applicable instruments

Fluorescence microscopy imaging systems, including fluorescence microscopy and filter sets suitable for DAPI (367/452), Green (495/517), and Orange (547/565).

Sample requirements

1. Applicable specimen type: paraffin embedded specimen of surgical resection or biopsy tissue.
2. The tissue in vitro should be fixed with 4% neutral formaldehyde fixative within 1 hour. After the tissue is fixed, it is often dehydrated and embedded in paraffin.

Sample pretreatment

It is recommended to use Cytowish's pretreatment reagent kit (catalogue number CW-PT-FISH).

Denaturation and hybridization

Your authentic partner in molecular cytogenetics

The following operations should be performed in a darkroom.

1. Take out the probe, let it stand at room temperature for 5min, turn it upside down with force, mix the probe well, centrifuge it briefly (do not vibrate with vortex apparatus), drop 10 μ l into the hybridization area of the cell drop, cover the 22mm \times 22mm cover glass immediately, the probe should be evenly spread under the cover glass without bubbles, and seal the edge with rubber (the edge sealing must be thorough to prevent the dry slide from affecting the test results in the hybridization process).
2. Place the glass slide in the hybridization instrument, denature at 85 $^{\circ}$ C for 5 minutes (the hybridizer should be preheated to 85 $^{\circ}$ C) and hybridize at 42 $^{\circ}$ C for 2 to 16 hours.

Washing

The following operations should be performed in a darkroom.

1. Carefully remove the sealing glue around the cover glass with tweezers to avoid sticking or moving the cover glass, immerse the sample in 2xSSC for about 5S, take it out, gently push a corner of the cover glass to the edge of the slide with tweezers, and gently remove the cover glass with tweezers;
2. Place the sample at 2XSSC room temperature for 1 min;
3. Take out the slides and immerse in a preheated at 68 $^{\circ}$ C 0.3% NP-40/0.4xSSC (Preparation of 0.3% NP-40/0.4xSSC: For 1L preparation, take 3mL NP-40 and 20mL 20xSSC, dissolve fully, mix well, and use 1M NaOH to adjust the pH to 7.2) solution and wash for 2min.
4. Take out the sample and immerse it in deionized water preheated at 37 $^{\circ}$ C in advance for 1min; dry it naturally in the dark place.

Counterstaining


The following operations should be performed in a darkroom.

10 μ L DAPI compound dye is dropped in the hybridization area of the glass slide and immediately covered. The suitable filter is selected for glass slide observation under the fluorescence microscope.

FISH results observation

Place the stained sections under a fluorescence microscope and the cells area is first confirmed under a low magnification objective (10 \times); under magnification objective (40 \times) a uniform cells distribution is observed; then the nucleus size uniformity, nuclear

boundary integrity, DAPI staining uniformity, no nuclei overlapping, cells clear signal are observed in the high magnification objective (60x, 100x).

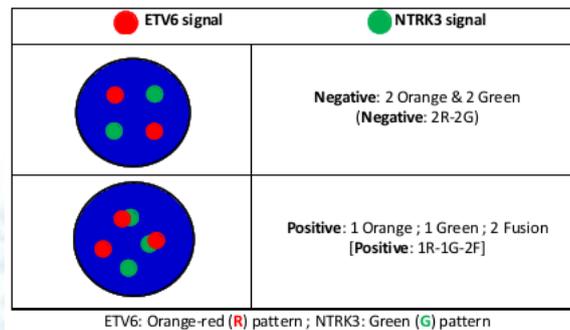

Test Method Limitations

1. The results of this kit will be affected by various factors of the sample itself, but also limited by hybridization temperature and time, operating environment, and limitations of current molecular biology technology, which may lead to erroneous results.
2. The user must understand the potential errors and accuracy limitations that may exist in the detection process.

Precautions

1. Please read this manual carefully before testing. The testing personnel shall receive professional technical training. The signal counting personnel must be able to observe and distinguish orange red and green signals.
2. When testing clinical samples, if it is difficult to count the hybridization signals and the samples are not enough to repeat the retest, the test will not provide any test results. If the amount of cells is insufficient for analysis, again, the test will not provide test results.
3. The formamide and DAPI counterstaining agent used in this experiment have potential toxicity or carcinogenicity, so they need to be operated in the fume hood and wear masks and gloves to avoid direct contact.
4. The results of this kit will be affected by various factors of the sample itself, but also limited by enzyme digestion time, hybridization temperature and time, operating environment and limitations of current molecular biology technology, which may lead to wrong results. The user must understand the potential errors and accuracy limitations that may exist in the detection process.

Your authentic partner in molecular cytogenetics

5. All chemicals are potentially dangerous. Avoid direct contact. Used kits are clinical wastes and should be properly disposed of.
6. This product is for clinical diagnosis and scientific research.



[Manuscript version and approval date]

Manual version: [V1.1 reviewed on 01 January 2020](#)

Approval date: [04 November 2019](#)