

## **YWHAE(17p13) Gene Break Apart Probe Detection Kit (CW-100)**

### **Intended use**

This kit performs fluorescence in situ hybridization staining on the basis of conventional staining, and provides auxiliary information for diagnosis for physicians. The test results are for clinical reference only and should not be used as the sole basis for clinical diagnosis. Clinicians should make comprehensive judgments on the test results based on factors such as the patient's condition, drug indications, treatment response and other laboratory test indicators.

### **Product composition**

The kit consists of YWHAE dual-color probe (100 $\mu$ L/Tube).

### **Storage condition**

Keep sealed away from light at  $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$ . The product is valid for 12 months. Avoid unnecessary repeated freezing and thawing that should not exceed 10 times. After opening, within 24 hours for short-term preservation, keep sealed at  $2-8^{\circ}\text{C}$  in dark. For long-term preservation after opening, keep the lid sealed at  $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$  away from light. This kit is shipped below  $0^{\circ}\text{C}$ .

### **Applicable instruments**

Fluorescence microscopy imaging systems, including fluorescence microscopy and filter sets suitable for DAPI (367/452), Green (495/517), and Orange (547/565).

### **Sample requirements**

1. Applicable specimen type: paraffin embedded specimen of surgical resection or biopsy tissue.
2. The tissue in vitro should be fixed with 4% neutral formaldehyde fixative within 1 hour. After the tissue is fixed, it is often dehydrated and embedded in paraffin.

### **Hybridization pretreatment**

Baking: Slides heating at  $80^{\circ}\text{C}$  for 30min or  $65^{\circ}\text{C}$  for 2h or overnight.

Dewaxing: According to the customer laboratory protocol (Commonly with Xylene for 15min).

## Your authentic partner in molecular cytogenetics

**Hydration:** Take out the slides and put them respectively into 100%, 85% and 70% EtOH at room temperature for 3 minutes each.

Take out the slides, and immerse them in deionized water for 3 minutes. Remove the excess of water on the slides by air-drying. **Permeation:** Immerse the slides in deionized water at 100°C and boil continuously for 20-40 minutes (Conventional 20min). Remove the excess of water on the slides by air-drying.

**Digestion:** Protease enzymic digestion at 37°C for 10-40 minutes. Mix the protease work buffer (50mmol HCl) and the 10x protease solution (Pepsin concentration 5%) in a proportion of 9:1 to prepare the enzymatic digestion solution.

**Washing:** Wash with 2xSSC at room temperature for 5 minutes.

**Dehydration:** Take out the slides and dehydrate in 70%, 85%, and 100% gradient ethanol at room temperature for 2 minutes each time. Remove the excess of EtOH solution on the slides by air-drying.

### **Denaturation and hybridization**

The following operations should be performed in a darkroom.

1. Take out the probe, let it stand at room temperature for 5min, turn it upside down with force, mix the probe well, centrifuge it briefly (do not vibrate with vortex apparatus), drop 10µl into the hybridization area of the cell drop, cover the 22mm×22mm cover glass immediately, the probe should be evenly spread under the cover glass without bubbles, and seal the edge with rubber (the edge sealing must be thorough to prevent the dry slide from affecting the test results in the hybridization process).
2. Place the glass slide in the hybridization instrument, denature at 85°C for 5 minutes (the hybridizer should be preheated to 85°C) and hybridize at 42°C for 2 to 16 hours.

### **Washing**

The following operations should be performed in a darkroom.

1. Carefully remove the sealing glue around the cover glass with tweezers to avoid sticking or moving the cover glass, immerse the sample in 2xSSC for about 5S, take it out, gently push a corner of the cover glass to the edge of the slide with tweezers, and gently remove the cover glass with tweezers;
2. Place the sample at 2XSSC room temperature for 1 min;
3. Take out the slides and immerse in a preheated at 68°C 0.3% NP-40/0.4xSSC (Preparation of 0.3% NP-40/0.4xSSC: For 1L preparation, take 3mL NP-40 and 20mL

## Your authentic partner in molecular cytogenetics

20xSSC, dissolve fully, mix well, and use 1M NaOH to adjust the pH to 7.2) solution and wash for 2min.

4. Take out the sample and immerse it in deionized water preheated at 37°C in advance for 1min; dry it naturally in the dark place.

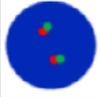
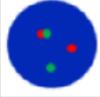
### Counterstaining

The following operations should be performed in a darkroom.

10µL DAPI compound dye is dropped in the hybridization area of the glass slide and immediately covered. The suitable filter is selected for glass slide observation under the fluorescence microscope.

### FISH results observation

Place the stained sections under a fluorescence microscope and the cells area is first confirmed under a low magnification objective (10x); under magnification objective (40x) cells distribution uniformity is observed; then the nucleus size uniformity, nuclear boundary integrity, DAPI staining uniformity, no nuclei overlapping, cells clear signal are observed in the high magnification objective (100x).

	● YWHAE gene site 5 signal	● YWHAE gene site 3 signal
	Negative: 2 fusion	
	Positive : 1 orange 1 green 1 fusion	

### Precautions

1. Please read this manual carefully before testing. The testing personnel shall receive professional technical training. The signal counting personnel must be able to observe and distinguish orange red and green signals.
2. When testing clinical samples, if it is difficult to count the hybridization signals and the samples are not enough to repeat the retest, the test will not provide any test results. If the amount of cells is insufficient for analysis, again, the test will not provide test results.

**Your authentic partner in molecular cytogenetics**

3. The formamide and DAPI counterstaining agent used in this experiment have potential toxicity or carcinogenicity, so they need to be operated in the fume hood and wear masks and gloves to avoid direct contact.
4. The results of this kit will be affected by various factors of the sample itself, but also limited by enzyme digestion time, hybridization temperature and time, operating environment and limitations of current molecular biology technology, which may lead to wrong results. The user must understand the potential errors and accuracy limitations that may exist in the detection process.
5. All chemicals are potentially dangerous. Avoid direct contact. Used kits are clinical wastes and should be properly disposed of.
6. This product is for clinical diagnosis and scientific research.

[Manuscript version and approval date]

Manual version: [V1.2 reviewed on December 07, 2021](#)

Approval date: [April 30, 2020](#)