

11q23.3/11q24.3 Gene Deletion Probe Detection Kit (CW-164)

Intended use

This kit performs in situ hybridization staining on the basis of conventional staining to provide physicians with auxiliary information for diagnosis. The test results are for clinical reference only and should not be used as the only basis for clinical diagnosis. Clinicians should make comprehensive judgment on the test results based on factors such as the patient's condition, drug indications, treatment response and other laboratory test indicators.

Product composition

The kit consists of two sets of probes: 11q23.3/CEP11 dual color probe and 11q24.3/CEP11 dual color probe as shown below.

Component name	Specifications	Quantity	Main components
11q23.3/CEP11 dual color probe	100µL/Tube	1	11q23.3 orange probe, CEP11 green probe
11q24.3/CEP11 dual color probe	100µL/Tube	1	11q24.3 orange probe, CEP11 green probe

Storage condition

This kit is shipped below 0°C. Keep sealed away from light at -20°C±5°C. The product is valid for 12 months. Avoid unnecessary repeated freezing and thawing that should not exceed 10 times. After opening, within 24 hours for short-term preservation, keep sealed at 2-8°C in dark. For long-term preservation after opening, keep the lid sealed at -20°C±5°C away from light.

Applicable instruments

Fluorescence microscopy imaging systems, including fluorescence microscopy and filter sets suitable for DAPI (367/452), Green (495/517), and Orange (547/565).

Sample requirements

1. Applicable specimen types: Surgical resection or paraffin-embedded biopsy specimens.
2. Isolated tissue should be fixed in vitro with 4% neutral formaldehyde fixative within 1 hour. After tissue fixation, regular dehydration and paraffin embedding should be performed.

Related reagents

1. 20×SSC, pH 5.3±0.2

Weigh 176g of sodium chloride and 88g of sodium citrate, dissolve in 800mL of deionized water, adjust the pH to 5.3±0.2 at room temperature, and complete to 1 L with deionized water. High-pressure steam sterilization, stored at 2-8°C, the solution shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

2. 2×SSC, pH 7.0±0.2

Take 100mL of the above 20xSSC, dilute with 800mL deionized water, mix, adjust the pH to 7.0±0.2 at room temperature, complete to 1L with deionized water, stored at 2-8°C, the shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

3. Ethanol Solution: 70% ethanol, 85% ethanol

Dilute 700ml, 850ml of ethanol with deionized water to 1L. The shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

4. 0.3% NP-40/0.4xSSC solution, pH 7.0-7.5

Take 0.6mL NP-40 and 4mL 20×SSC, add 150mL deionized water, mix, adjust the pH to 7.0-7.5 at room temperature, with deionized water complete to a volume of 200mL. Stored at 2-8°C, the shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

5. Fixation solution (methanol: glacial acetic acid = 3:1)

Prepare a ready to use fixation solution by mixing thoroughly 30ml of methanol and 10ml of glacial acetic acid.

6. 0.075M KCl solution

Weigh 2.8g of potassium chloride, dissolve in 400mL of deionized water and complete to 500mL with deionized water. Stored at room temperature, the solution shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

7. Diamidinyl phenylindole (DAPI) counterstain

Use commercially available anti-queenching DAPI counterstain.

Pretreatment

It is recommended to use Cytowish's pretreatment reagent kit (catalogue number CW-CT-FISH).

Denaturation and hybridization

The following operations need to be carried out in the darkroom.

1. Take the probe at room temperature for 5 minutes. Briefly centrifuge manually (do not use vortex or shaker instrument). Take 10µl droplet in the cell and drop in the hybridization zone, immediately cover 22mmx22mm glass slide area; spread evenly without bubbles the probe under the glass slide covered area and seal edges with rubber (edge sealing must be thorough to prevent dry film from affecting the test results during hybridization).
2. Place the glass slide in the hybridization instrument, denature at 85°C for 5 minutes (the hybridizer should be preheated to 85°C) and hybridize at 42°C for 2 to 16 hours.

Washing

The following operations should be performed in a darkroom.

1. Take out the hybridized glass slides, remove the rubber on the coverslip and immediately place the slides into 2xSSC for 5 seconds, and gently remove the coverslip.
2. Place the glass slides in 2xSSC at room temperature.
3. Remove and immerse the slides in a 0.3% NP-40/0.4xSSC solution preheated at 68°C for 2 min.
4. Immerse the glass slides in deionized water at 37°C for 1min, and dry naturally in the dark.

Counterstaining

The following operations should be performed in a darkroom.

10µL DAPI compound dye is dropped in the hybridization area of the glass slide and immediately covered. The suitable filter is selected for glass slide observation under the fluorescence microscope.

FISH results observation

Place the stained sections under a fluorescence microscope and the cells area is first confirmed under a low magnification objective (10×); under magnification objective (40×) a uniform cells distribution is observed; then the nucleus size uniformity, nuclear

boundary integrity, DAPI staining uniformity, no nuclei overlapping, cells clear signal are observed in the high magnification objective (60x, 100x).

 11q23.3 signal	 CEP11 signal
	Negative: 2 Orange & 2 Green (Negative: 2R-2G)
	Positive: n Orange ; 2 Green [Positive: nR-2G], n≥3
11q23.3: Orange-red (R) pattern; CEP11: Green (G) pattern	
 11q24.3 signal	 CEP11 signal
	Negative: 2 Orange & 2 Green (Negative: 2R-2G)
	Positive: 1 Orange ; 2 Green [Positive: 1R-2G]
11q24.3: Orange-red (R) pattern; CEP11: Green (G) pattern	

Precautions

1. Please read this manual carefully before testing. The testing personnel shall receive professional technical training. The signal counting personnel must be able to observe and distinguish orange red and green signals.
2. When testing clinical samples, if it is difficult to count the hybridization signals and the samples are not enough to repeat the retest, the test will not provide any test results. If the amount of cells is insufficient for analysis, again, the test will not provide test results.
3. The formamide and DAPI counterstaining agent used in this experiment have potential toxicity or carcinogenicity, so they need to be operated in the fume hood and wear masks and gloves to avoid direct contact.
4. The results of this kit will be affected by various factors of the sample itself, but also limited by enzyme digestion time, hybridization temperature and time, operating environment and limitations of current molecular biology technology, which may lead to wrong results. The user must understand the potential errors and accuracy limitations that may exist in the detection process.
5. All chemicals are potentially dangerous. Avoid direct contact. Used kits are clinical wastes and should be properly disposed of.
6. This product is for clinical diagnosis and scientific research.



[Manuscript version and approval date]

Manual version: [V1.2 reviewed on 07 December 2021](#)

Approval date: [30 April 2020](#)