

1q21 and 1p32 Gene Anomaly Probe Detection Kit (CW-197)

Intended use

The kit uses orange fluorescein-labeled 1p32(CDKN2C), and green fluorescein-labeled 1q21(CKS1B), and bind 1p32/1q21 gene with the target site by in situ hybridization.

Product composition

The kit consists of 1q21 orange-red probe (100 μ L/Tube).

Storage condition

Keep sealed away from light at $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$. The product is valid for 12 months. Avoid unnecessary repeated freezing and thawing that should not exceed 10 times. After opening, within 24 hours for short-term preservation, keep sealed at $2-8^{\circ}\text{C}$ in dark. For long-term preservation after opening, keep the lid sealed at $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$ away from light. The kit is transported below 0°C .

Applicable instruments

Fluorescence microscopy imaging systems, including fluorescence microscopy and filter sets suitable for DAPI (367/452), Green (495/517), and Orange (547/565).

Sample requirements

1. Applicable specimen types: Fresh bone marrow samples that have not been fixed are stored at 4°C for less than 24 hours; bone marrow cells suspension after fixation is stored at -20°C for less than 6 months; the prepared bone marrow cell slides can be set at -20°C for less than 1 month.
2. When specimens are stored at too high or too low a temperature (eg, frozen), the specimen will not be used for testing and should be discarded.
3. If the cells suspension is excessively volatile or contaminated during storage, the sample should be discarded.

Related reagents

1. **20 \times SSC, pH 5.3 \pm 0.2**

Weigh 176g of sodium chloride and 88g of sodium citrate, dissolve in 800mL of deionized water, adjust the pH to 5.3 ± 0.2 at room temperature, and complete to 1 L with deionized water. High-pressure steam sterilization, stored at $2-8^{\circ}\text{C}$, the solution shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

2. 2×SSC, pH 7.0±0.2

Take 100mL of the above 20×SSC, dilute with 800mL deionized water, mix, adjust the pH to 7.0 ± 0.2 at room temperature, complete to 1L with deionized water, stored at $2-8^{\circ}\text{C}$, the shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

3. Ethanol Solution: 70% ethanol, 85% ethanol

Dilute 700ml, 850ml of ethanol with deionized water to 1L. The shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

4. 0.3% NP-40/0.4×SSC solution, pH 7.0-7.5

Take 0.6mL NP-40 and 4mL 20×SSC, add 150mL deionized water, mix, adjust the pH to 7.0-7.5 at room temperature, with deionized water complete to a volume of 200mL. Stored at $2-8^{\circ}\text{C}$, the shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

5. Fixation solution (methanol: glacial acetic acid = 3:1)

Prepare a ready to use fixation solution by mixing thoroughly 30ml of methanol and 10ml of glacial acetic acid.

6. 0.075M KCl solution

Weigh 2.8g of potassium chloride, dissolve in 400mL of deionized water and complete to 500mL with deionized water. Stored at room temperature, the solution shelf life is of 6 months. Discard if the reagent appears cloudy (turbid) or contaminated.

7. Diamidinyl phenylindole (DAPI) counterstain

Use commercially available anti-quinching DAPI counterstain.

Sample pretreatment

1. Sample collection: Take 3mL of anticoagulated bone marrow cell samples.
2. Cell harvesting: Place 3 mL of anticoagulated peripheral blood in a 15 mL centrifuge tube, centrifuge at 300 g for 5 min, carefully discard the supernatant, and resuspend about 500 μL of the residue.

Your authentic partner in molecular cytogenetics

3. Cell washing: Add 5 mL of 1×PBS buffer, mix and resuspend the cell pellet, centrifuge at 300 g for 5 min, carefully discard the supernatant, and resuspend the cells with about 500µL of the residue; repeat 1 time.
4. Cells hypotonicity: Add 10mL of hypotonic solution pre-warmed to 37°C and place in a water bath at 37°C for 10-15min.
5. Cells pre-fixation: Pre-fix the cells by adding 1mL (10% by volume) of fixative solution to the cell suspension after the completion of hypotonic osmosis. Gently pipette, mix and centrifuge for 5 min at 300 g, discard the supernatant, and resuspend about 500µL of the residue.
6. Cell fixation: Slowly add 10mL of fixative solution to the cell suspension at room temperature for 10 min, centrifuge at 300 g for 5 min, and resuspend the cells with about 500µL of the residue; repeat once (the cells may be fixed several times until the cells pellet is washed and cleaned).
7. Cell suspension preparation: Pipet the supernatant and add the appropriate amount of fixative solution to prepare the appropriate cell suspension concentration.
8. Slides preparation: Pipet 3-10µl of cell suspension drop onto the slides, put at 56°C for 30min.

Slides pretreatment

1. At room temperature, rinse the glass slides twice with 2XSSC solution for 5min each time.
2. Place the glass slides in 70% ethanol, 85% ethanol and 100% ethanol and dry for 2 minutes.

Denaturation and hybridization

The following operations need to be carried out in the darkroom.

1. Take out the probe, leave it at room temperature for 5min, turn it upside down with force, mix it well, and then centrifuge it for a short time (no vortex instrument vibration). Take 10µl drop in the cell hybridization area, and immediately cover the cover glass of 22mm×22mm. The probe should be evenly expanded under the cover glass without bubbles, and then seal the edge with rubber glue (the edge must be completely sealed to prevent the dry piece from affecting the test results in the hybridization process).
2. Place the glass slides in the hybridization instrument, denature at 88°C for 2 minutes (the hybridizer should be preheated to 88°C) and hybridize at 45°C for 2 to 16 hours.

Washing

The following operations should be performed in a darkroom.

1. Take out the hybridized glass slides, remove the rubber on the coverslip and immediately place the slides into 2xSSC for 5 seconds, and gently remove the coverslip.
2. Place the glass slides in 2xSSC at room temperature.
3. Remove and immerse the slides in a 0.3% NP-40/0.4×SSC solution preheated at 68°C for 2 min.
4. Immerse the glass slides in deionized water at 37°C for 1min, and dry naturally in the dark.

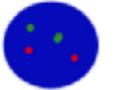
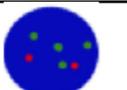
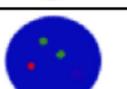
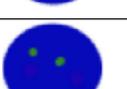
Counterstaining

The following operations should be performed in a darkroom.

10µL DAPI compound dye is dropped in the hybridization area of the glass slide and immediately covered. The suitable filter is selected for glass slide observation under the fluorescence microscope.

FISH results observation

Place the counterstained film under the fluorescence microscope, and first put it under the low-power objective lens (10x). Confirm the cell area under the microscope; Go to 40x under the objective lens, find a position where the cells are evenly distributed; Then in the high-power objective (100x) The FISH results of nuclei were observed.

 1p32 (CDKN2C)	 1q21(CKS1B)
	Negative: 2 Fusions
	Positive: CKS1B amplification
	Positive: CKS1B deletion
	Positive: CDKN2C deletion

Your authentic partner in molecular cytogenetics

Precautions

1. Please read this manual carefully before testing. The testing personnel shall receive professional technical training. The signal counting personnel must be able to observe and distinguish orange red and green signals.
2. When testing clinical samples, if it is difficult to count the hybridization signals and the samples are not enough to repeat the retest, the test will not provide any test results. If the amount of cells is insufficient for analysis, again, the test will not provide test results.
3. The formamide and DAPI counterstaining agent used in this experiment have potential toxicity or carcinogenicity, so they need to be operated in the fume hood and wear masks and gloves to avoid direct contact.
4. The results of this kit will be affected by various factors of the sample itself, but also limited by enzyme digestion time, hybridization temperature and time, operating environment and limitations of current molecular biology technology, which may lead to wrong results. The user must understand the potential errors and accuracy limitations that may exist in the detection process.
5. All chemicals are potentially dangerous. Avoid direct contact. Used kits are clinical wastes and should be properly disposed of.
6. This product is for clinical diagnosis and scientific research.

[Manuscript version and approval date]

Manual version: [V1.2 reviewed on 07 December 2021](#)

Approval date: [01 February 2019](#)